

COVID-19 Antigen Test

Clinical Sensitivity and Specificity Study Report

Drafted by: Nicolas Ma

Approved by: Jerry Zheng

Final report date: 20200910

Management of the study: Artron Laboratories Inc.

**R& D Department
Quality Control Department**

Table of Contents

Title Page

Table of Contents

Study Summary

1. Purpose
2. Reference and Compliance
3. Materials
4. Experiment Design
5. Evaluation Criteria
6. Result
7. Conclusion
8. Report

Study Summary

The purpose of this study was to obtain accurate information regarding the clinical performance Sensitivity and Specificity of Artron COVID-19 Antigen Test. The clinical evaluation was carried out for the clinical performance of Artron COVID-19 Antigen Test.

1. Purpose

To validate the clinical performance sensitivity and specificity of Artron COVID-19 Antigen Test.

2. Reference and Compliance

FDA guidance for In vitro diagnostic medical device

The present study conformed to all applicable laws and regulations.

3. Materials

- COVID-19 positive specimens were confirmed by RT-PCR. And non-COVID-19 Nasal/Nasopharyngeal Swabs from non-febrile and non-respiratory patients and confirmed non-COVID-19 by RT-PCR.
- Artron COVID-19 Antigen Test, Lot: COV19-Ag-S-1

4. Study Design:

4.1 The clinical performance was evaluated in RT-PCR confirmed COVID-19 infected Nasopharyngeal swab specimens and non-COVID-19 infected Nasopharyngeal Swabs specimens from subjects in the chosen hospitals and clinical laboratories. All the samples should be tested with Artron COVID-19 Antigen Test.

4.2 Test conditions:

- All tests were performed by the clinical technicians in the clinical laboratory according to the manufacturer's instructions using the confirmed samples.
- Visual interpretations of the results of COVID-19 Antigen Test were made independently by the clinical technician.
- The testing center was responsible to summarize the result.

5. Evaluation Criteria

Positive (POS): Both C and T lines appear regardless of color intensity.

Negative (NEG): Only C line appears.

Indeterminable (IND): C line appears; T line is so weak that it cannot be determined as positive.

Invalid result (/): Neither C line nor T line appears.

6. Clinical Results

6.1 Clinical Site One

Results from SANTA ANA HOSPITALS, Manila, Philippines: Evaluation has been done in SANTA ANA HOSPITALS.

The clinical evaluation was carried out for the clinical performance of Artron COVID-19 Antigen Test. Total 38 nasopharyngeal swabs from COVID-19 infected patients and 133 non-COVID-19 infected patients were tested.

Table 1 Artron COVID-19 Antigen Test results

		Results of COVID 19 Ag test		Subtotal
		Positive	Negative	
RT-PCR	Positive	37	1	38
	Negative	0	133	133
Subtotal		37	134	171

Artron COVID 19 Ag Test kit could detect the Nasopharyngeal swabs with high clinical performance sensitivity and specificity listed below:

Sensitivity: $37/38 = 97.3\%$

Specificity: $133/133 = 100.0\%$

Total Agreement: $(37+133)/171 = 99.4\%$

Total 38 Nasopharyngeal swabs from SARS-Cov-2 infected patients, and 133 non-SARS-Cov infected patients were tested. Among all of those chosen samples, Artron COVID-19 Ag test kit identified out total 37 SARS-Cov-2 positive from 38 SARS-Cov-2 infected patients' samples; the clinical sensitivity was 97.3%, no false positive from total 133 non-SARS-Cov infected cases, which demonstrated the specificity was 100 %, total agreement is 99.4%

6.2 Clinical Site Two

Results from Clinical lab: Experiments have been done in local Hospitals/Labs in Ahmedabad, Gujarat, India.

The clinical evaluation was carried out for the clinical performance of Artron COVID-19 Antigen Test. Total 56 Nasopharyngeal swabs from COVID-19 infected patients and 68 non-COVID-19 infected patients were tested.

Table 2 Artron COVID-19 Antigen Test results

		Results of COVID 19 Ag test		Subtotal
		Positive	Negative	
RT-PCR	Positive	48	8	56
	Negative	0	68	68
Subtotal		48	76	124

Artron COVID 19 Ag Test kit could detect the Nasopharyngeal swabs with high clinical performance sensitivity and specificity listed below:

Sensitivity: $48/56 = 85.7\%$

Specificity: $68/68 = 100.0\%$

Total Agreement: $(48+68)/124 = 93.5\%$

Total 56 Nasopharyngeal swabs from SARS-Cov-2 infected patients, and 68 non-SARS-Cov infected patients were tested. Among all of those chosen samples, Artron COVID-19 Ag test kit identified out total 48 SARS-Cov-2 positive from 56 SARS-Cov-2 infected patients' samples; the clinical sensitivity was 85.7 %, no false positive from total 68 non-SARS-Cov infected cases, which demonstrated the specificity was 100 %, total agreement is 93.5%

7. Conclusion

The sensitivity and specificity of Artron COVID-19 Antigen Test to the positive Nasopharyngeal swab specimens were calculated based on the detection results of RT-PCR. The Artron COVID-19 Antigen test sensitivity >85.7% and specificity is >99.0% respectively, the overall agreement >93.5 %.

8. Report

8.1 Original raw data is archived at Quality Control Department

8.2 The original final report is archived in Quality Control Department.